

Diversatek Healthcare CleanFreak® Endoscope Transport Pad

Instructions for Use.
Read carefully prior to use.

INSTRUCTIONS FOR USE:

1. Remove pad from bag and place on cart.
2. Unfold pad to cover cart surface with cinch strings and absorbent section facing up.
3. After procedure, place coiled scope on center of pad.
4. Grasp strings and pull to cinch around scope for transport.
5. Dispose of properly.

DESCRIPTION OF SYMBOLS USED ON LABELS



Non-Sterile Device



Do Not Re-Use



Do Not Use if Package is Damaged



Not Made with Natural Rubber Latex



Date of Manufacture (YYYY-MM-DD)

Diversatek Healthcare, Inc.
102 E Keefe Ave.
Milwaukee, WI 53212 USA
P 800.558.6408 | 414.265.7620
sales@diversatekhc.com
DiversatekHealthcare.com

Made in the U.S.A.
CleanFreak® is a registered trademark of Diversatek Healthcare, Inc.
Diversatek™ is a trademark of Diversatek, Inc.