



Medovations CleanFreak® Endoscope Transport Pad

*Instructions for Use.
Read carefully prior to use.*

DIRECTIONS FOR USE:

1. Remove pad from bag and place on cart.
2. Unfold pad to cover cart surface with cinch strings and absorbent section facing up.
3. After procedure, place coiled scope on center of pad.
4. Grasp strings and pull to cinch around scope for transport.
5. Dispose of properly.

DESCRIPTION OF SYMBOLS USED ON LABELS



Non-Sterile Device



Do Not Re-Use



Do Not Use if Package is Damaged



Not Made with Natural Rubber Latex



Date of Manufacture (YYYY-MM-DD)

Medovations, Inc.
102 E Keefe Ave.
Milwaukee, WI 53212 USA
800.558.6408
414.265.7620
Fax 414.265.7628
medo@medovations.com
www.medovations.com

Made in the U.S.A.
*CleanFreak® and Medovations® are registered trademarks of
Medovations, Inc.*