




HOBBS MEDICAL, INC.

Precision Instruments for Endoscopy

Active Cords Monopolar - Reusable

INSTRUCTIONS FOR USE

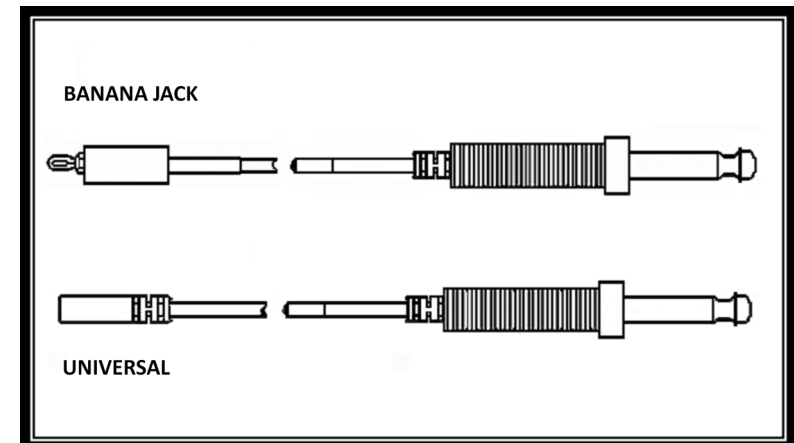

EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands



HOBBS MEDICAL, INC.

8 Spring Street

Stafford Springs, CT 06076 USA

salesdept@hobbsmedical.com

Tel. +1.860.684.5875


www.hobbsmedical.com

COM-1042 REV H

READ CAREFULLY BEFORE USING

SYMBOLS

The meanings of the symbols shown on the package and/or this instrument are as follows:

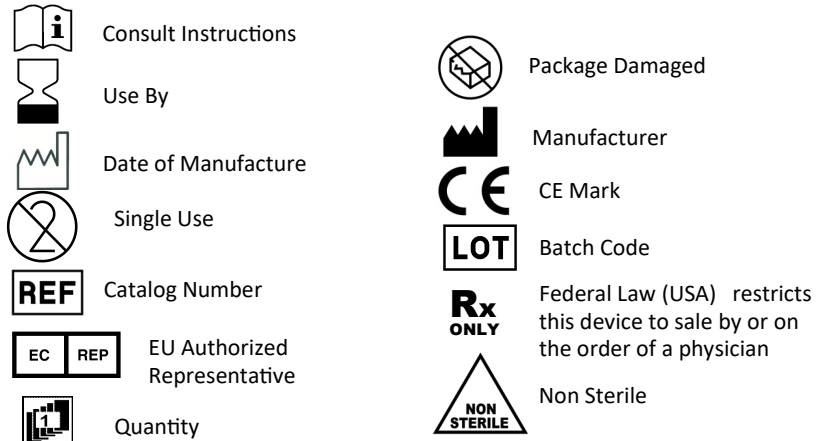

SIGNAL WORDS

The following signal words are used throughout this manual:

WARNING

Indicates a potentially hazardous situation, which if not avoided, could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation, which if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices or potential equipment damage.

SAFETY

Indications for Use: To be used with Endoscopic electrosurgical accessories in conjunction with an approved electrosurgical unit.

WARNING

Do not use this device for any purpose other than the stated indication for use.

WARNING

The user must thoroughly understand the technical principles, clinical techniques and risks associated with the use of high frequency electrosurgical energy in general and the characteristics of their chosen electrosurgical generator in particular before using this product.

CONSTRUCTION

Design: The Hobbs Medical Reusable Active Cords are designed to allow the delivery of monopolar high frequency energy to electrosurgical accessories.

CONTRAINDICATIONS

None Known

INSTRUCTIONS FOR USE

1. Remove the device from the package. **Save the product lot number for future reference.**
2. Inspect for any functional abnormality. If any irregularities are noted, call Hobbs Medical for a return authorization number.
3. The Active Cord should snap snugly into the electrosurgical unit receptacle, as well as the electrosurgical device handle fitting.
4. At the completion of the procedure, detach Active Cord from device and remove from the electrosurgical unit.

CLEANING

Active cords should be wiped down with facility approved disinfectant to prevent cross contamination.

WARNING

Discard Active Cord if insulated cord appears compromised or has any loose fittings.

For further information: Please contact Hobbs Medical